

Logo Care Instructions

Thank you for choosing an OuterFactor® Custom LED UV printed case. To ensure the longevity and vibrancy of the logo, please follow the care instructions below.

Cleaning:

When cleaning the logo area, choose cleaning solutions that are appropriate for the specific use case. The case should be cleaned with the **lowest potency solution in accordance with the standards relevant to your application.**

- **Wipe the logo area using smooth-textured towels or wipes** with a compatible cleaning solution.
- **Do not saturate the logo area** during cleaning. Excessive moisture for extended periods may affect the logo's durability.
- Promptly **dry the logo area** with a clean, dry cloth for the best life of the logo. Avoid letting moisture soak on the surface for extended periods.
- Avoid submerging the logo in solution.

In hospital settings specifically:

- *Use appropriate cleaning solutions suitable for the specific patient care area where the tablet is utilized to wipe the logo area. It is recommended that you disinfect the entire tablet case first, wiping the logo area last. Optional cleaning solutions include but are not limited to:*
 - Cavi Wipes
 - Bleach wipes
 - Up to 70% isopropyl alcohol
 - Super Sani-Cloth® Germicidal Disposable Wipe

Avoid Abrasives:

- When cleaning, **avoid using abrasive cleaning mechanisms such as scrubbing when possible.**
- **Avoid using abrasive textures such as rough-textured towels or sponges, as the logo is most vulnerable during cleaning / while wet.**
- Instead, **use smooth-textured towels or wipes**, such as microfiber cloths, lint-free cloths, or other suitable smooth textured cleaning wipes.

Drying:

After cleaning, **allow the logo area to dry fully before returning the case to its regular use.** You can facilitate this by wiping the logo with a clean, dry cloth if too much solution is applied. If the logo is wiped with an appropriate amount of cleaning solution, drying should take 5-10 seconds following the wipe.

If you have any questions or concerns, please do not hesitate to contact us for further assistance. Thank you for choosing OuterFactor!